

DIALYSIS CATHETER CLEANING AGENTS

CHRONIC CATHETERS

CLEANING AGENT	Tandem-Cath™ 10 Fr Dual Lumen Catheter System	PermCath™ Chronic Catheter	MAHURKAR™ 14.5 Fr Chronic Catheter	Palindrome™ Precision 14.5 Fr Chronic Catheter	Palindrome™ Precision H- Heparin Coated Catheter SI- Silver Ion Coated Catheter HSI- Heparin & Silver Ion Coated Catheter	Palindrome™ Precision RT- Reverse Tunnel Catheter
Alcavis™ 50 (fka Amukin 50%) disinfectant	YES	YES	YES	YES	YES	YES
Aqueous-based povidone iodine	YES	YES	YES	YES	YES	YES
Bacitracin ointment	YES	YES	YES	YES	YES	NO
Bactroban™ cream	NO	YES	YES	YES	YES	NO
Chlorhexidine (Hibiclens™) antiseptic	YES	YES	YES	YES	YES	YES
Chloraprep™ antimicrobial solution	NO	YES	YES	YES	YES	YES
ExSept™ (sodium hypochlorite) antiseptic	YES	YES	YES	YES	YES	YES
Hydrogen peroxide	YES	YES	YES	YES	YES	YES
Isopropyl alcohol 70%	NO	YES	YES	YES	YES	YES
Neosporin™ antibiotic	YES	YES	YES	YES	YES	NO
Shur-Clens™ cleansing solution	NO	YES	YES	YES	YES	YES
Octenisept	NO	NO	NO	YES	NO (SI, HSI) YES (H)	NO

Please Note: Mixing of these solutions has not been tested and is not recommended.

DIALYSIS CATHETER CLEANING AGENTS

ACUTE CATHETERS

CLEANING AGENT	MAHURKAR™ Elite		
	12 Fr Dual Lumen Catheter 13.5 Fr HF Dual Lumen Catheter 12.5 Fr Triple Lumen Catheter	MAHURKAR™ 8 Fr Dual Lumen Catheter 10 Fr Dual Lumen Catheter	MAHURKAR™ 11.5 Fr Dual Lumen Catheter 12 Fr HP Triple Lumen Catheter 13.5 Fr HF Dual Lumen Catheter
Alcavis™ 50 (fka Amukin 50%) disinfectant	YES	YES	YES
Aqueous-based povidone iodine/PVP solution	YES	YES	YES
Aqueous chlorhexidine gluconate	YES	YES	YES
Bacitracin ointment	YES	YES	YES
Bactroban™ cream	YES	YES	YES
Betadine™ povidone iodine	YES	YES	YES
Chlorhexidine (Hibiclens™) antiseptic	YES	YES	YES
Chloraprep™ antimicrobial solution	YES	YES	YES
ExSept™ cleanser	YES	YES	YES
Hydrogen peroxide	YES	YES	YES
Isopropyl alcohol 70%	YES	NO	NO (12 Fr, 11.5 Fr) YES (13.5 Fr)
K-Y™ jelly	YES	YES	YES
Neosporin™ antibiotic	YES	YES	YES
Octenisept™ disinfectant	YES	NO	NO
Triple Polysporin™ antibiotic ointment	YES	NO	NO

Please Note: Mixing of these solutions has not been tested and is not recommended.

PERITONEAL CATHETERS

CLEANING AGENT	Argyle™ Peritoneal Dialysis Catheters (ALL)
Alcavis™ 50 (fka Amukin 50%) disinfectant	YES
Aqueous-based povidone iodine	YES
Bacitracin ointment	YES
Bactroban™ cream	YES
Chlorhexidine (Hibiclens™) antiseptic	YES
Chloraprep™ antimicrobial solution	YES
ExSept™ (sodium hypochlorite) antiseptic	YES
Hydrogen peroxide	YES
Isopropyl alcohol 70%	NO
K-Y™ jelly	YES
Neosporin™ antibiotic	YES